ATLANT

OPTICAL – SUN

EN Product name

Optical eyeglass frame. In accordance with EU Regulation 2017/745 (Annex VIII, Chapter III: Classification Rules) and Standard EN ISO 12870:2016, this eyeglass frame belongs to the category of Class I medical devices. The main purpose of the product is to be used with ophthalmic lenses fitted according to a doctor's prescription for the correction of visual defects.

Categories of sunglass filters according to ISO 12312-1:2013/A1:2015.

CATEGORY 0

Sunglasses with very light tint. Very limited glare reduction. Limited UV protection.

CATEGORY 1

Lightly tinted sunglasses. Limited protection against glare. Moderate UV protection.

CATEGORY 2

General-purpose sunglasses. Good protection against glare. Good UV protection.

CATEGORY 3

General-purpose sunglasses. High protection against glare. Good UV protection.

CATEGORY 4

Very dark glasses for special use. Very high glare reduction. Very high protection against extreme light (sea, snow, high mountains or desert). Good UV protection.

OFFVIEW uses only high-quality lenses that are comfortable and safe. They provide clear vision and high absorption of UVA and UVB rays. The choice of lens type depends on the level of ambient light and individual sensitivity. A selection of models includes polarized, photochromic and A.R. filters, which provide greater protection against solar glare.

Material from which the product is made

Stainless steel (and its alloys), acetate, ALYTE®, titanium and combinations of these.

Warnings

Optical frames are supplied with presentation lenses, which are not intended for use but only to maintain the shape of the glasses and to allow the frame to be tried on under conditions as close as possible to the real configuration.

For glasses made of ALYTE® material, do not use products containing alcohol, as they may deteriorate the material and affect its resistance and surface finish.

Allergies

Our glasses are made from high-quality hypoallergenic materials that strictly comply with all current safety and health regulations. However, it cannot be excluded that certain more sensitive or allergic individuals may experience adverse reactions as a result of contact with the materials and treatments applied to the frames.

Use

Only use frames in perfect condition. When using the glasses, avoid sudden movements. Always put on and remove the glasses with both hands. Otherwise, the glasses may deform or even break. Do not wear the glasses on top of your head, as the temples of the frame may deform and increase the distance between them. Protect your glasses from extreme temperatures (above +35°C and below -10°C), radiant heat, chemicals (for example cleaning products, acetone, glue, ethers, alcohols, etc.), cosmetics (for example perfume, hair spray) and seawater. Do not use in the sauna or solarium.

Transport and storage

When not using the glasses, always store them in a suitably sized rigid case. Place the glasses in the case with the lenses facing upward.

The case must be in perfect condition, as worn lining or dirt may scratch the glasses. Do not place the glasses on the dashboard or in the glove compartment. We recommend not using mineral lenses in the car.

Cleaning

Eyeglass frames may be sensitive to chemical agents (for example acidic or alkaline cleaners, solvents, alcohols, etc.). Clean the frame with a dry microfiber cloth or with a neutral/special cleaning liquid. In case of occasional contact with agents such as salt water (sea water), pool water, chemicals such as cosmetics, hair spray, sunscreen and insect repellent, immediate cleaning with a soft damp cloth is recommended. Ultrasonic cleaning once a year is advisable. This process removes acidic deposits, thus preventing the risk of allergic skin symptoms and corrosion of the device.

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